# Advanced Instruments

## OsmoTECH® HT

Calibration and Verification Standard Set

REF	Code	Description
	3MA000	Two 2mL ampules of 0 mOsm/kg H <sub>2</sub> O
	3MA003	Two 2mL ampules of 300 mOsm/kg H <sub>2</sub> O
	3MA085	Two 2mL ampules of 850 mOsm/kg H <sub>2</sub> O
	3MA200	Two 2mL ampules of 2000 mOsm/kg H <sub>2</sub> O
	3MA029	Two 2mL ampules of 290 mOsm/kg H <sub>2</sub> O

### Intended use

Calibration and Verification standards are used to calibrate and verify calibration on your OsmoTECH HT Plate Based Micro-osmometer. Refer to the instrument User's Guide for calibration instructions.

### Ingredients

These standards are prepared from salts. 2-Hydroxybiphenyl Sodium Salt Tetrahydrate, 0.01%, is added as a preservative.

### Warning

Carefully snap off the top of the ampule. There may be sharp edges at breakpoint, and could result in injury if caution is not taken.

### Instructions for use

- Carefully snap off the top of the ampule.
- Load the ampules in the calibration plate, follow the onscreen instructions to perform calibration.

### Storage and handling

Follow these storage and handling guidelines for optimal results:

- Dispose of any opened ampules at the end of the testing day.
- Do not freeze.

	Storage	Stability
Unopened	2 - 30 °C (36 - 86 °F)	Refer to expiration date stamped on the box

### Limitations

If there is visible evidence of microbial growth in the ampules, do not use the calibration standards. Erroneous results can occur from adverse shipping and/or storage conditions, use of expired materials, or sample handling errors.

Do not use the product if the primary packaging has been damaged. Discard the product immediately.

### Interpreting results

Please visit [aicompanies.com/expectedranges](http://aicompanies.com/expectedranges) for expected values and expected ranges based on your osmometer model.

**NOTE:** If results fall outside the total expected range, it may indicate unsatisfactory calibration, operator error, contamination of reagents, or faulty performance of equipment.

### Disposing of materials

Handle this product according to established good laboratory practices, using appropriate precautions. Dispose of materials according to your institution's practices. Discard all materials in a safe and acceptable manner that is in compliance with all country, state and local requirements.

### For sales and service

Contact your Advanced Instruments Distributor.

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WMDE B.V.